## MedFem Menstrual Cup User Guide

Thank You for purchasing MedFem Menstrual Cup and for taking one step forward towards a better health of you and environment.

### IMP PRODUCT SAFETY INFORMATION:

**PLEASE DO NOT USE the MedFem Menstrual Cup** before you have read and understood these instructions. It is NOT inserted like tampons, or diaphragms.

### Instructions about the MedFem Menstrual Cup:

#### How to Insert the MedFem Menstrual Cup?

Before first use, soak the MedFem Menstrual Cup in a vessel filled with hot water for 5-10 minutes or put it in an open pot of boiling water for 5-10 minutes. Please make sure there is enough water in the pan so you don't burn out the cup. Use caution while inserting

Fig. 1

Fig. 2

a tight "U" shape; OR

• **Fold Option 2** (Fig. 2) Place a finger on the top rim of the cup and press it down into the center of the inside base to form a triangle. This makes the top rim much smaller to insert.

**Step 3:** Hold With one hand, hold the folded sides firmly together.

#### Step 4: INSERT / WEAR:

Take a comfortable position standing, sitting on the toilet or squatting. Relax your vaginal muscles. Take a few deep breaths if you are not feeling relaxed as a first time

3

and removing cup with long fingernails. When first trying The MedFem Menstrual Cup, you may require a bit of a practice and you may take 2-3 cycles to feel comfortable with insertion, positioning and removal.

**STEP 1: Wash and Sterilize the MedFem Menstrual Cup:** Prior to insertion, thoroughly wash your hands and sterilize The MedFem Menstrual Cup for 5-10 min with hot water or a mild, oil and fragrance-free soap. If you wash it with soap, make sure you rinse it in water thoroughly so no traces of soap remain on the cup. **DO NOT STERILISE IN ANTI SEPTIC LIQUID AS IT MAY HAVE SIDE EFFECTS.** Wear Menstrual Cup immediately after sterilizing.

#### Step 2: Fold the MedFem Menstrual Cup

• **Fold Option 1** (Fig. 1) Press the sides of the cup together and then fold it in half again forming

2

user. Gently separate your labia with your free hand. Insert MedFem Menstrual Cup into the vagina, horizontally tilted towards your tailbone. Continue to insert until the stem is even with the vaginal opening and does not protrude.

You can also make sure the cup is completely open by inserting a finger into the vagina and gently pressing on one side of the vaginal wall to create space for the cup to fully open.

If the stem irritates the vaginal opening once it is positioned correctly, you may remove the cup and trim the stem slightly.

#### How to remove the MedFem Menstrual Cup

Thoroughly wash your hands as recommended. In a comfortable position, either standing, sitting on the toilet, or squatting with your buttocks resting on your

4

heels, bear down in a series of gentle downward pushes with your abdominal/pelvic floor muscles (as if you are having a bowel movement).

Insert your thumb and index finger until you can reach the stem. Firmly pinch the base of the cup. Squeeze the cup gently to release the seal. As you remove it, angle it sideways while moving it from side to side. Once removed, hold the Cup upright so the contents remain inside the cup.

**NOTE:** For first time users, remain calm if you cannot reach the stem and are unable to remove it. Follow removal instructions. Contract and flex your vaginal muscles in such a way like you are pushing a baby at the time of vaginal birth.

Do not use any foreign object to assist with removal as this may cause damage to the cup

5

or vaginal canal. In a rare case scenario, if you are unable to remove the Menstrual Cup at all, seek medical advice from a physician.

#### Empty, Clean, Sanitize and Reuse:

MedFem Menstrual Cup can be worn up to a maximum of 10-12 consecutive hours even overnight. MedFem Menstrual Cup conveniently holds one ounce (25 ml). By monitoring the fullness of the cup, you will quickly learn how often to empty it according to your personal needs. We recommend that it is emptied, washed and rinsed a minimum of 2-3 times daily.

Clean Menstrual Cup with warm water, or use a mild, oil and fragrance-free soap. Do not use vinegar, tea tree oil, scented soap, and castile/peppermint soap, oil-based soap, rubbing alcohol, antibacterial soap, hand sanitizer, pre-moistened wipes, dish soap,

6

hydrogen peroxide, bleach or harsh chemicals as these may damage the silicone and cause irritation.

The MedFem Menstrual Cup may naturally, slightly discolor over time. This does not impair its functionality. As needed, it may be boiled for 5-10 minutes in an open pot with plenty of water. Do not leave the boiling pot unattended. If you accidentally burned your cup may be ruined and should be replaced immediately.

The tiny holes located below the rim must be kept clean at all times. Gently stretch each hole under warm running water to remove any debris. At the end of your cycle, thoroughly wash, rinse and dry MedFem Menstrual Cup completely and store it in the cotton bag provided, not in a plastic bag or airtight container.

7

It is not necessary to remove MedFem Menstrual Cup before urinating or having a bowel movement.

If you are unable to wash MedFem Menstrual Cup after removal (for instance when using a public restroom), wash your hands thoroughly before entering the stall, empty the contents in the toilet and simply use a dry or damp tissue to clean the cup. At the next convenient time, clean as instructed. When traveling, or otherwise, always wash the cup using potable (safe to drink) water.

**What should you do if the cup leaks, moves up, slides down and is difficult to insert or remove, or is uncomfortable:**

Please review instructions. For a proper fit, ensure that it is positioned horizontally and rotates easily.

8

#### How often to replace a menstrual cup:

Silicone is very durable but depending on factors unique to each woman, the life span of the MedFem Menstrual Cup may vary. Inspect your cup regularly for signs of deterioration, such as a sticky or powdery film, severe discoloration or odor. Since it is a menstrual hygiene product, a general guideline is to replace it if the MedFem Menstrual Cup was used while you experienced a yeast or bacterial infection.

#### Precautionary Warnings:

- Use pursuant to directions and during menstruation only.
- Remove the menstrual cup prior to intercourse. Menstrual cup is not a contraceptive device and it does not protect you against sexually transmitted infections.

9

• **MedFem Menstrual Cup and IUDs:** All women are different so be informed about your particular IUD's benefits and risks and consult with your physician.

- Do not use a menstrual cup if you have a yeast or bacterial infection. Resume use only after the infection has completely treated.
- Do not flush the menstrual cup down the toilet.
- Keep menstrual cup away from children and animals.
- If you experience any adverse effects, discontinue use and consult with your physician.

#### RETAIN FOR FUTURE REFERENCE

This User Guide is informational in nature and is not offered as medical advice, nor does it substitute for a consultation with your

10

physician. If you have any gynecological/ medical concerns or conditions, please consult your physician prior to using a MedFem Menstrual Cup.

#### Customer Service:

Carefully follow the instructions for best results. Visit [www.medfem.in](http://www.medfem.in) for more information. For other questions, contact our trained Consumer Care Team by phone or email.

#### ATTENTION:

MedLife Foundation cares for your safety. Menstrual cups are personal hygiene medical devices and may not be returned or exchanged. All sales are final. MedFem Menstrual Cup has registered trademarks of Medlife Pharma Healthcare Pvt. Ltd. This is the official User Guide for MedFem

11

Menstrual Cup and content may change without prior notice. Visit [www.medfem.in](http://www.medfem.in) for up to date information.

#### Marketed By:

Medlife Pharma Healthcare Pvt. Ltd.,  
968, Medlife House, At Post - Bahal,  
Tal. Chalisgaon, Dist. Jalgaon-424 101

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